

Passy Muir Valves

All Passy Muir® Valves

- Have the patented closed position “no leak” design
- Are designed to fit the universal 15mm hub of disposable tracheostomy tubes including neonatal and pediatric tubes
- Can be used interchangeably by tracheostomized and ventilator dependent patients with the exception of the PMV® 2020



**PMV® 007**  
(Aqua Color®)  
Designed for ventilator application with disposable tubing



**PMV® 2000**  
(clear)  
Low profile design with lower resistance, recommended for outpatient use



**PMV® 2001**  
(Purple Color®)  
Low profile design with lower resistance, recommended for inpatient use



**PMV® 005**  
(white)  
David's Original Design

Passy Muir Valve O<sub>2</sub> Accessory

Passy Muir® Valve Oxygen Adapter PMA® 2000

The PMA® 2000 Oxygen Adapter snaps onto the PMV® 2000 and PMV® 2001 and allows for easy delivery of low flow supplemental oxygen and humidity.



PMA® 2000 Adapter shown with PMV® 2001\*

PMV® 2001 (Purple Color®)  
PMA® 2000




\* PMV sold separately

Optional Tether

PMV® Secure-It®

The Secure-It® strap is designed for use with the lower resistance 2000 series valves and attaches the valve to the trach tie to prevent loss.

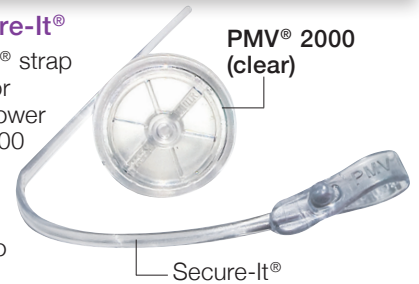


Passy Muir Valves for Metal Tubes

PMV® 2020 (clear)

The PMV® 2020 (clear) with the PMA® 2020-S adapter are for use with metal Jackson Improved Tracheostomy Tubes (Sizes 4, 5, and 6).

PMV® 2020 (clear)

PMA® 2020-S

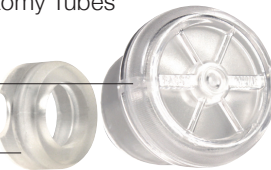

PMA® 2020-S




PMV® 2020 (clear)

Jackson Improved Tracheostomy Tube

PMV® 2001 (Purple Color®)

The Jackson Improved Tracheostomy Tube with Permanent 15mm Adapter can accommodate all Passy Muir® Valves except the PMV® 2020.

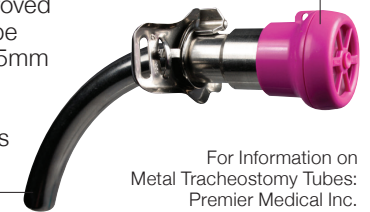

For Information on Metal Tracheostomy Tubes:  
Premier Medical Inc.  
www.premusa.com  
888-670-6100

Passy Muir Adapters

Ventilator Adapters

PMV® AD1522 Adapter

Part # PMV-AD1522  
15mm x 22mm Adapter



PMV® AD22 Adapter

Part # PMV-AD22  
22mm Silicone Adapter



Adapters for Duckbill Style Tracheostomy Tubes\*

DB15™ Adapter\*

Part # PM-DB15  
15mm Adapter



DigiSil™ Adapter\*

Part # PM-OCLN  
15 mm Silicone Adapter  
Includes Secure-It®



These adapters provide a secure connection between the Passy Muir Valve and ventilator tubing, closed suction systems, or other adapters.

\* Not for use with mechanical ventilation

Passy Muir Filters & HMEs

Airway Protection Filter\*

Part # PM-APF15  
Tracheostomy Viral & Bacterial Airway Protection Filter



Heat Moisture Exchanger\*







Part # PM-HME  
Heat Moisture Exchanger






\* Not a Passy Muir Valve

To order, or for more information about any of our products, call 800.634.5397 or visit www.passymuir.com

Ventilator Connections

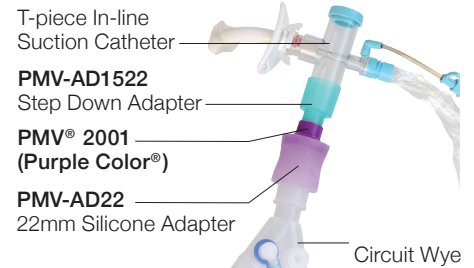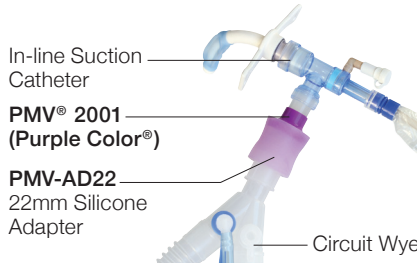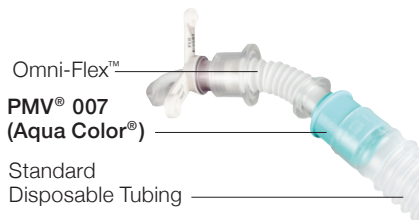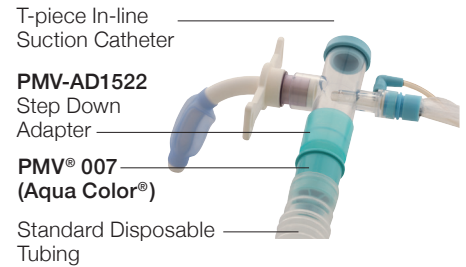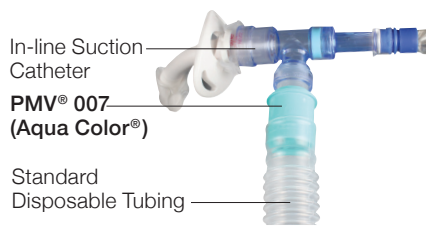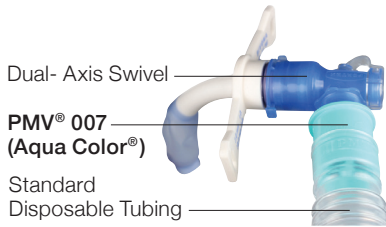

Pediatric Ventilator Connections

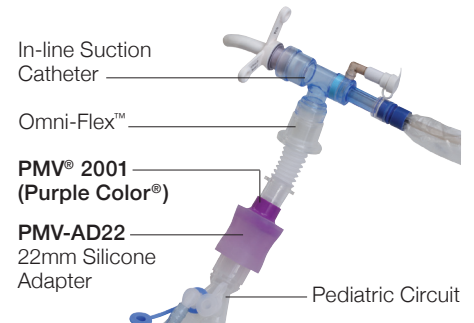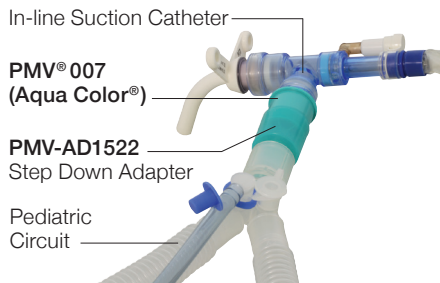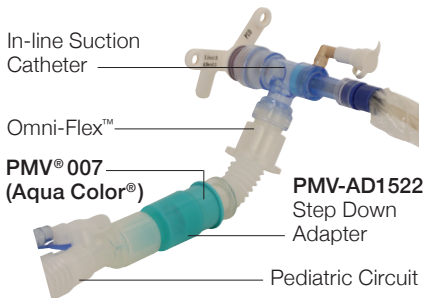

Care & Cleaning of the PMV®

Daily Cleaning Procedures

1. Swish valve in pure soap and warm water.
2. Rinse valve thoroughly in warm running water.
3. Allow valve to air dry thoroughly before placing it in storage container.
4. DO NOT use hot water, peroxide, bleach, vinegar, alcohol, brushes, or cotton swabs to clean valve.



Education & Clinical Support



Speech and Respiratory Clinical Specialists are available to answer your questions

1.800.634.5397



Email your questions

info@passymuir.com

Educational Presentations



- Facility Locations
- State and National Conferences
- Society Meetings
- Seminars
- Colleges and Universities



- Live Special Events Webinars
- Self-Study Webinars
- Free CEU Courses
- Free Clinical Resources

www.passymuir.com



PASSY-MUIR PRODUCTS PROUDLY MADE IN USA



To order, or for more information about any of our products, call 800.634.5397 or visit www.passymuir.com