




GdYU_]b[' J U`j Y`I gY`k]h\`
HfUW\ Ycghca miUbX`
A YW\ Ub]W\` J Ybh]`Uh]cb.`
Bck ž`K YEY`HU`_]b[°

DfYgYbhYX`]b`&\$&6


© 2026 Passy-Muir, Inc.

These materials are protected by the Copyright laws of the United States and may not be copied, distributed, or republished by the recipient. Recipients are licensed to use the materials for their personal use as an attendee of this seminar.

Passy-Muir®, PMV®, the purple valve®, and the aqua valve™ are trademarks of Passy-Muir, Inc.

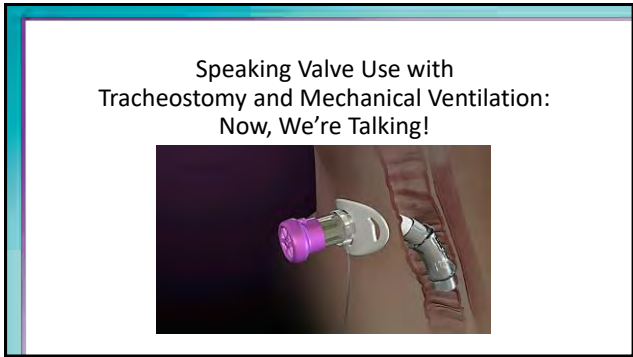


1

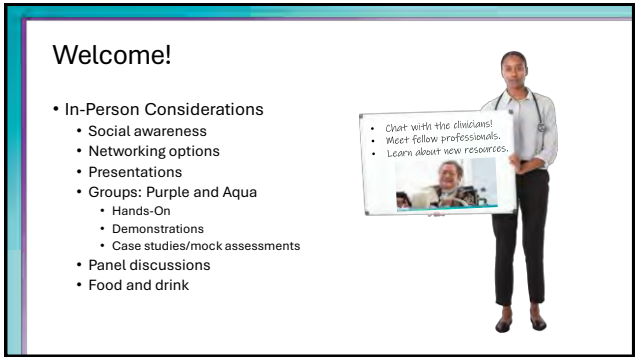


2



3

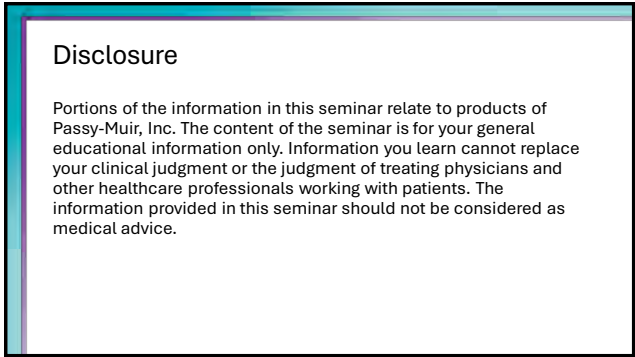

4

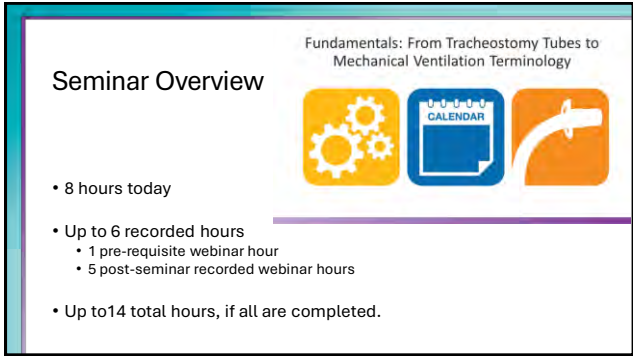

5



6

slido




What is the actual color of the PMV 007?


Start presenting to display the poll results on this slide.

7

Assessment and Placement: Non-Ventilator




8

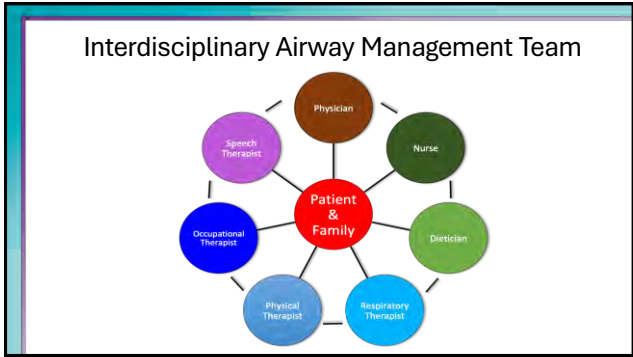

9

Begins in the ICU: Effects of Bed Rest




- The negative impact of bed rest is well known
- No evidence supports efficacy of bed rest
- Disuse atrophy at the cellular level begins within 4 hours of implementing bed rest
- Healthy adults, bed rest¹
 - Strength declined by 1 – 1.5% per day
 - Mood changes
 - Loss of coordination, balance and work tolerance
 - Casting: Strength declines by 25% in 7 day²

Griffiths et al. Nutrition 1995; 11:428-432
De Jonghe et al. CCM 2000; 5:309-315

10


Physiologic Changes after Tracheostomy

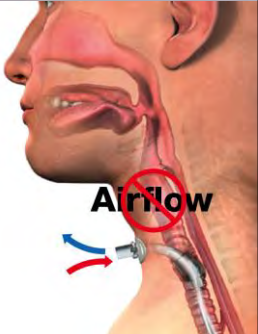

Respiration: Patient inhales and exhales through open trach tube. No airflow past inflated cuff

11


Physiologic Changes after Tracheostomy

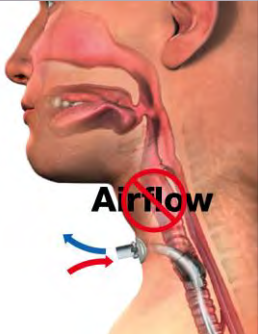

- Speech
- Smell
- Taste
- Sensation
- Reduced positive airway pressure
 - Poor secretion management
 - Reduced cough

12


Cuff Management or Clinical Complications Happen



- Esophageal impingement
- Backflow
- Necrosis and trauma
- Laryngeal tethering
- Late complications
 - Granuloma - stenosis
 - Tracheomalacia
 - Fistulae

13


Application of the PMV: Non- Ventilator



14


Researchers: Medical errors now third leading cause of death in United States

such as communication breakdowns when patients are handed off from one department to another.




Published: May 3, 2016

15



16

Reminder: Passy-Muir Valve




PMV 2001 (Purple color™)

- Bias-closed
- No-leak

What are the benefits of the Valve?

17




What is a benefit of using the Passy Muir Valve?

Start presenting to display the poll results on this slide.

18

Benefits of Closing the System with PMV

- Restores normal physiology
- Reconnects the upper and lower airway
- Providing a closed system
 - Communication
 - Smell and taste
 - Secretion management
 - Sensation
 - Cough
 - Swallowing
 - Positive airway pressure
 - Quality of life



19

Psychological Benefits


- Communication with family
- Participation in decision making
- Reduced sense of isolation/ anxiety
- Better sense of well-being
- Communication with caregivers

*Freeman-Sanderson, A. L., Tozler, L., Elkins, M. R., & Kenny, B. (2018). Quality of life improves for tracheostomy patients with return of voice: A mixed methods evaluation of the patient experience across the care continuum. *Intensive Critical Care Nursing*, 48, 01-06. doi:10.1016/j.iccn.2018.02.004
*Freeman-Sanderson, A. L., Tozler, L., Elkins, M. R., & Phipps, P. R. (2016). An intervention to allow early speech in ventilated tracheostomy patients in an Australian intensive care unit (EIC): A randomized controlled trial. *Respiratory Critical Care*, 20(2), 124. doi:10.1016/j.rescc.2015.12.012
*Freeman-Sanderson, A. L., Tozler, L., Elkins, M. R., & Phipps, P. R. (2015). Quality of life improves with return of voice in tracheostomy patients in intensive care: An observational study. *Journal of Critical Care*, 30, 148-154. doi:10.1016/j.jcc.2014.03.012
*Freeman-Sanderson, A. L., Tozler, L., Elkins, M. R., & Phipps, P. R. (2015). Return of voice for ventilated tracheostomy patients in ICU: A randomized, controlled trial of early-targeted intervention. *Critical Care Medicine*, 43(6), 1075-1081. doi:10.1097/CCM.0000000000000163

20

Impact on PEEP


- Closed System vs Open
 - Improved gas exchange
 - Improved oxygen saturation levels
 - Decreased risk of atelectasis
- "My patient is not tolerating cuff deflation trials"



21

Initiating the Assessment: Team Approach


- Have a plan: Who does what
- Block off the time
- Education
- Reassure the patient
- Perform good oral care
- Suctioning as needed
- Body position and posture
- Position of head, neck, and tracheostomy tube



22

Patient Selection


- Awake and alert
- Medically stable
- Complete cuff deflation
- Manageable secretions
- Patent upper airway




23

Checklist: Take Baseline Measurements


- Oxygenation
- Vital signs
- Breath sounds
- Color
- Work of breathing
- Patient responsiveness




24

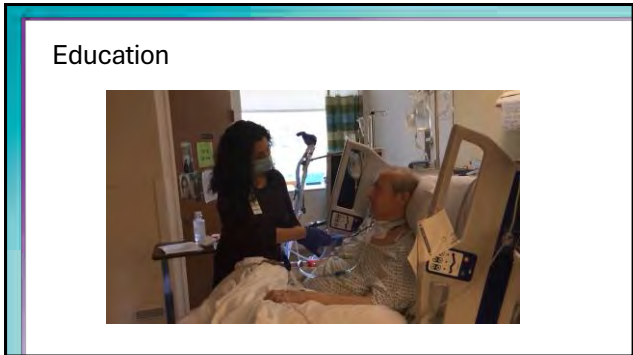


25

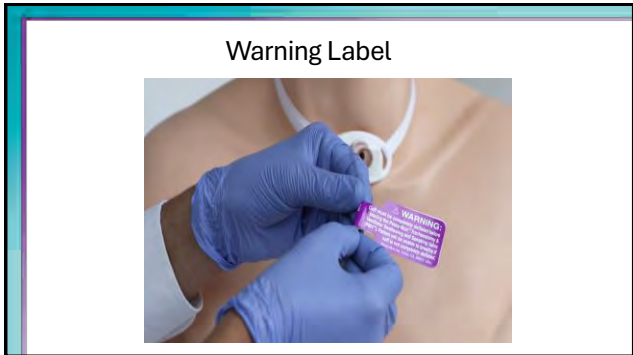


26

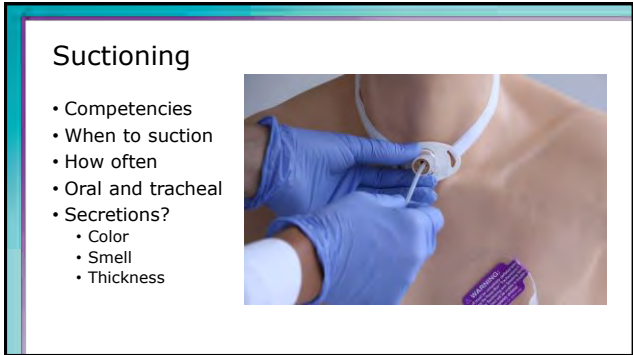


27

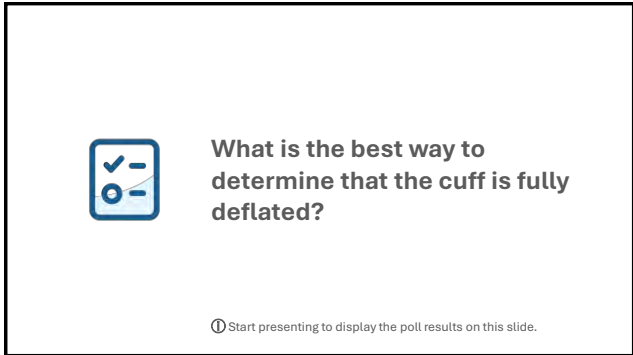

28

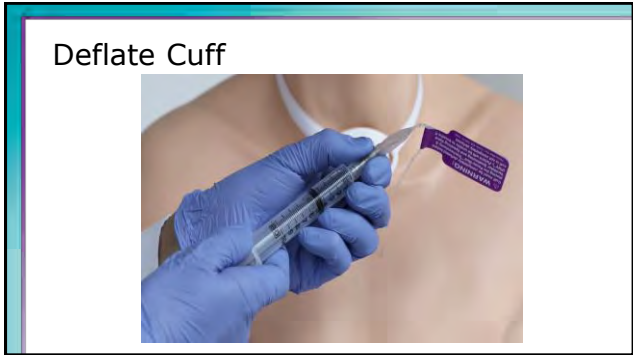


29

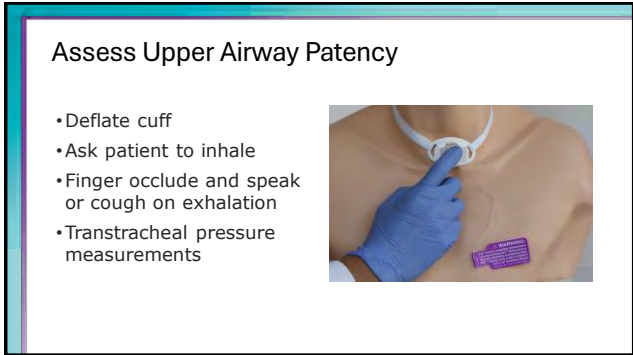

30


Assessment for Placement

- Transtracheal pressure measurements
 - Back pressure
 - Air trapping
 - Assessing for patent upper airway




31

PMV Placement

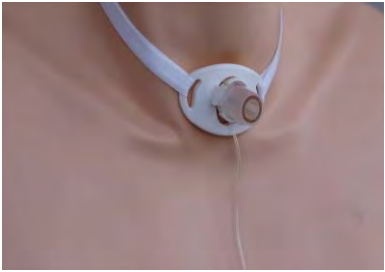


32

Airflow After Tracheotomy




Airflow with Closed System

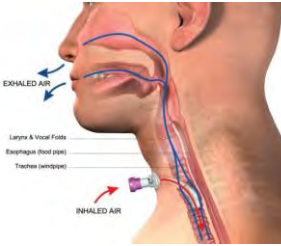


33

Initial Placement: Coughing



34


Initial Placement: Voicing



35

Advantages of a Closed Respiratory System vs Open Tracheostomy

- Open tracheostomy
 - Reduced airflow
 - Reduced positive airway pressure
 - Reduction in the pressurized system
- Closed Respiratory System
 - Allows graded exhalation and pressure regulation
 - Feeding and Swallowing
 - Posture and balance
 - Upper extremity force/strength



36

Pressurized System

- Restored or improved pressurized system:
 - Intraoral
 - Subglottic pressure
 - Respiratory – PEEP
 - Esophageal ??
 - Intrathoracic
 - Respiratory
- Leads to improved:
 - Feeding and swallowing
 - Cough and throat clear
 - Trunk support and postural control
 - Respiratory function



37


More Education



38

Care and Cleaning

- Average lifetime of 2 months

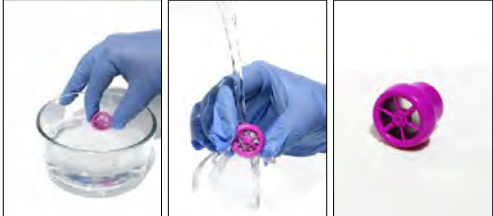

39

Decannulation: Troubleshooting and Interventions



40

Pre and Post Decannulation



41


In Summary: What has been accomplished?

- Early intervention:
 - Avoid disuse atrophy
- Close the system to improve:
 - Phonation: access to vocal communication
 - Sensation and secretion management: cough and throat clear
 - Taste and smell
 - Time to weaning and decannulation




42

Thank you!
Any questions?




43

Breakout Sessions:
Tracheostomy Tubes and PMVs, Cuff
Management, and
Mock Assessments




44

Tracheostomy Tubes and PMVs




45

Cuff Management




46

Mock Assessments




47

Mock Assessment One




48



What were some signs of distress that the patient was showing?

① Start presenting to display the poll results on this slide.

49




What are possible causes of back pressure?

① Start presenting to display the poll results on this slide.

50


Factors Affecting Expiratory Air Flow

- Size or type of tracheostomy tube
- Presence and degree of obstruction
- Edema
- Secretions
- Incomplete cuff deflation
- Tube position



51


Airway Obstruction




52

**Troubleshooting:
Downsize or Different Brand Tube**

Trach A




Trach B




53

Resolution




54




What was the resolution?

55

Mock Assessment Two: Low Level




56




What are your primary considerations with a low-level patient?

57




58




What could be some causes of her back pressure?


59




60

 **What would be your next step for therapy?**

61

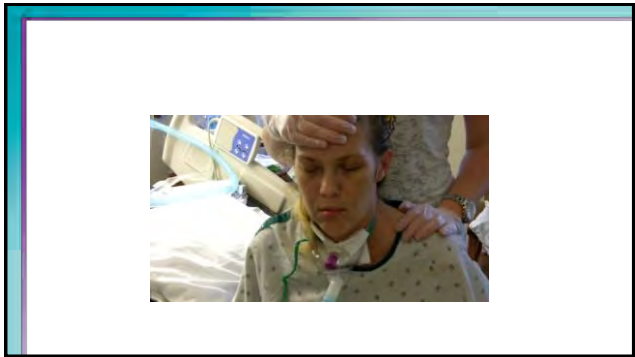


62

 **Why is using a PMV during ammonia testing beneficial?**

63


Troubleshooting Expiratory Air Flow

- Excessive coughing?
- No voice?
- Anxiety?




64

Troubleshooting and Treatment




65

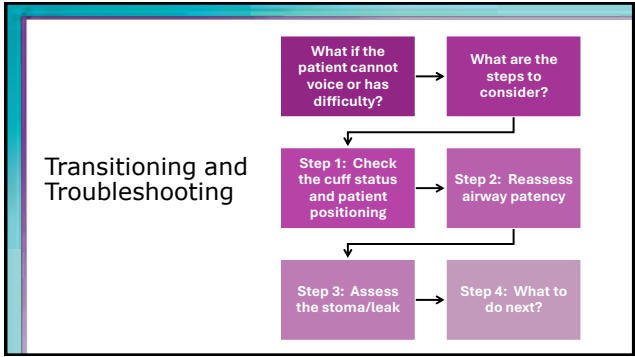


66

Voice?

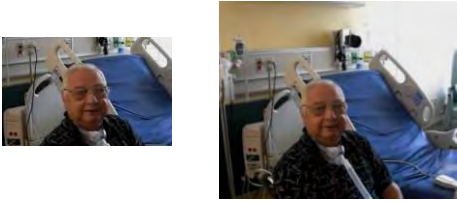


67

Secretions?



68

Cuff issue?

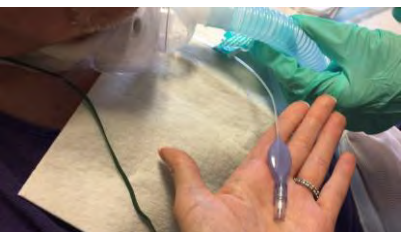

69


Questions to Determine Therapy

- What is diagnosis?
- Why do they have difficulty with:
 - Voice?
 - Breath support?
 - Language and/or cognition?
 - Dysarthria?
- What about swallowing?

70

Wear Time

- Patient specific
 - Patient's cognitive status
 - Medical needs
- Minutes to hours
- Treatment plan




71

Goals for treatment

- Wear time
 - Patient will wear the speaking Valve:
 - For ___ minutes to improve communication.
 - During ___-minute therapy session without need for Valve removal.
 - For ___-hour periods of time while awake.
- Other goal areas that impact wear time
 - Participate in conversation with audible voicing on ___ out of ___ sentences.
 - Complete ___ number of RMT tasks while wearing the speaking Valve.

72

Break: 15 Minutes (Lunch)




73

LUNCH



& LEARN

74



Audience Q&A

75

Basics of Ventilator Application of the No-leak Valve



1

Can Patients Drink While Ventilated?



2


Indications for Invasive Mechanical Ventilation

- Can no longer support with NIV
- Airway protection
- Hypercapnic respiratory failure
- Hypoxemic respiratory failure
- Cardiovascular distress
- Anticipated patient decline or impending transfer



3

Positive Pressure Ventilation

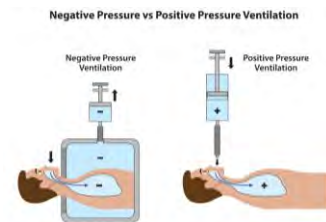

4

Invasive Ventilation


- Usually requires airway to be sealed with little to no leak present.
- Seal is achieved with a cuff at the end of the artificial airway.




5

Invasive Ventilation

- Endotracheal Tube
- Tracheostomy Tube

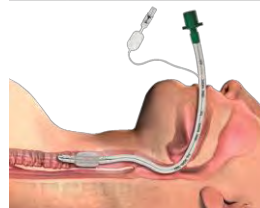


6




A drive through settings & modes of ventilation

7

Mode=who's driving?




- How is the breath controlled:
 - Full control
 - Assist control
 - Support only



8


Mandatory vs. Spontaneous Breathing

- **Mandatory breaths:**
 - Given by the vent at a set rate
- **Spontaneous breath:**
 - Started by the patient




9


Settings=How the drive feels



- **Rate (RR)**-breaths per minute (like cruise control speed)
- **Tidal Volume (VT)**-size of each breath
- **Pressure**-force pushing air in
- **PEEP**- small amount of pressure left in the lungs at the end of exhalation, so they don't collapse
- **FiO₂** - percent of oxygen in the air given (room air is ~21%)



10

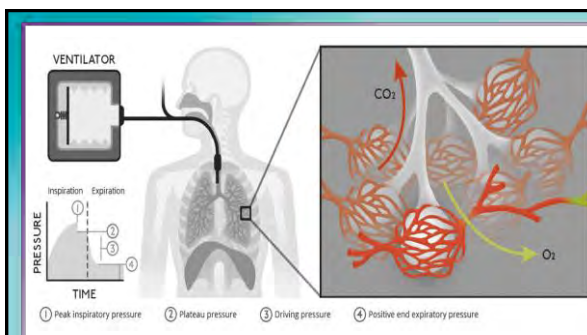


11

Conventional vs. Non-Conventional Ventilation

Conventional Ventilation	Non-Conventional Ventilation
<ul style="list-style-type: none"> • Follows a normal inhalation exhalation (normal breathing) • Closely mirrors how one breathes without support 	<ul style="list-style-type: none"> • Does not follow normal breathing patterns • Used when lungs are damaged - air is changed to protect them

12




Modes of Ventilation

Compatible with PMV Use:

- A/C, VC, & PC
- SIMV
- CPAP/PS
- NIV

13



Modes of Ventilation


Not recommended with PMV:

- PRVC
- AVAPS
- NAVA
- Other "auto" adjusted modes


14

PRVC

- Vent delivers a set tidal volume
- Pressure adjusts automatically to achieve the target tidal volume
- Combines the benefits of volume and pressure control



15




Measured Ventilator Parameters


- Exhaled Tidal Volume
- Exhaled Minute Volume
- Peak Inspiratory Pressure (PIP)
- Total Respiratory Rate
- Mean Airway Pressure
- PEEP

16

Can Patients Eat While Ventilated?



17

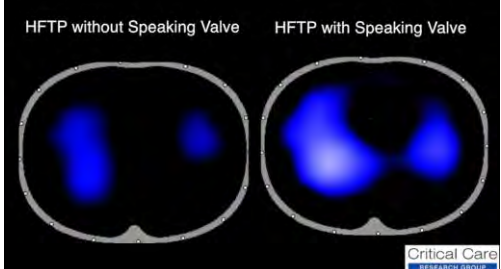

(Sutt et al. (2016). *Critical Care*. 20:91)

Critical Care RESEARCH GROUP

18

Steps For In-line Valve Placement

19

Why use a Passy-Muir Valve with patients who are mechanically ventilated?

- Verbal communication
- Improved lung recruitment and diaphragm involvement
- More rapid weaning from the ventilator
 - Rehabilitation tool
- Improved secretion management
 - More effective cough
 - Reduces need for suctioning
- Improves quality of life



20



21

Step 1: Assessment

22

Patient Selection Criteria



- Awake and alert
- Hemodynamically stable
- Able to manage complete cuff deflation
- Manageable secretions
- Patent Airway

23

Establish Baseline: Assess Vital Signs and Work of Breathing

- Oxygenation
- Vital Signs
- Breath sounds
- Color
- Work of breathing
- Patient responsiveness



24

Assess Ventilator Parameters

Three parameters that give you the general state of your patient's respiratory status:

1. FiO_2
2. PEEP
3. PIP



25

Assess Ventilator Parameters



FiO_2

- Fraction of inspired Oxygen
- Room Air 21%
- Supplemental $O_2 > 21\%$

26

Assess Ventilator Parameters



PEEP

- Positive End-Expiratory Pressure
- Extra pressure left in the lungs at the end of exhalation that stents the alveoli open
- PEEP and FiO_2 work together to improve oxygenation

27

Assess Ventilator Parameters



PIP (Peak Inspiratory Pressure)

- The max amount of pressure to deliver volume
- Sum of the inspiratory pressure required to deliver volume + PEEP
- PIP indicates the compliance of the lungs

28

"Must Know" for PMV Use

- $FiO_2 \leq .50$
- $PEEP \leq 10 \text{ cmH}_2O$
- $PIP \leq 40 \text{ cmH}_2O$
- VTi & VTe
- Patient stability and ability to manage secretions and tolerate cuff deflation



29

Step 2: Patient Preparation and Education

30

Team Approach

- Timing and tube selection
- Introducing a speaking valve
- When to downsize
- Plan of care
- Decannulation
- Impacts continuity of care
- Impacts safety, length of stay, and cost



31

Patient Preparation

- Body position and posture
- Position of head, neck, and tracheostomy tube



32

Pre-Placement, General Observations, and other Considerations

- Have a plan and block time
 - Pick a good time of the day
 - Reduce noise and interference
- Education
 - Reassure the patient
- Address pain issues
- Position the patient




33

Step 3: Assess For Airway Patency

34

Airway Patency Assessment With Mechanical Ventilation


- Requires complete cuff deflation
- Assess the leak or airflow into the upper airway
- Use vent parameters to determine airway patency

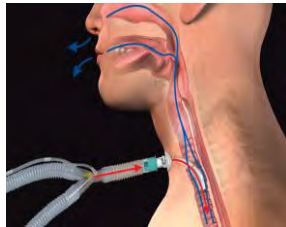

35

Cuff Deflation and Mechanical Ventilation

- Set parameters do not change when cuff is deflated
- Cuff deflation generates less resistance to flow
- Ventilatory system is no longer sealed, there is a leak

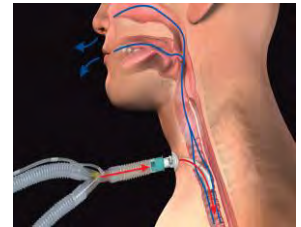

36

Settings in Volume Ventilation



- VT
- RR/f
- PEEP
- Inspiratory Time
- Trigger Sensitivity
- FiO₂
- Alarms

37


Ventilator Assessment

- Note Vent Settings:
 - Set Vt
 - PEEP
 - RR/f
 - FiO₂
- Note Vent Measurements
 - PIP
 - Exhaled Vt (Vte)
 - Total RR



38

VC: Patient Assessment

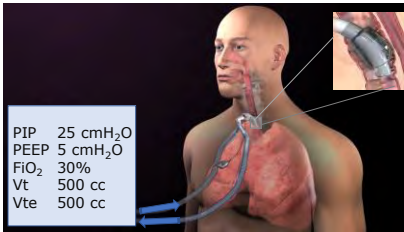

39

Upper Airway Patency Assessment

- Turn Down PEEP
 - PEEP down by 5
- Then,
 - Slow cuff Deflation




40

Upper Airway Patency

Cuff Inflated-Closed Circuit

Cuff Deflated-Open Circuit

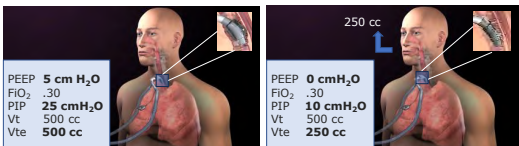

41

Upper Airway Patency Assessment



42

Settings in Pressure Ventilation

- Inspiratory Pressure
- RR/f
- PEEP
- Inspiratory Time
- FiO_2
- Trigger Sensitivity
- Alarms



43


Ventilator Assessment

- Note Vent Settings:
 - Set IP
 - PEEP
 - RR/f
 - FiO_2
- Note Vent Measurements
 - PIP
 - Exhaled Vt (Vte)
 - Total RR



44

PC: Patient Assessment

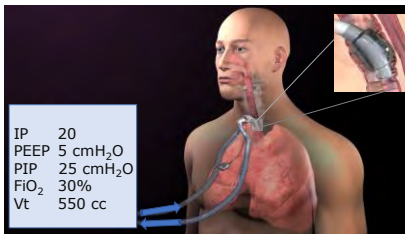

45

Upper Airway Patency Assessment

- Turn Down PEEP
 - PEEP down by 5
- Then,
 - Slow cuff Deflation



46

Upper Airway Patency

Cuff Inflated-Closed Circuit Cuff Deflated-Open Circuit



47

Step 4: Assemble the Necessary Parts & Pieces

48

Ventilator Connections



49

In-line Placement of the PMV® 007 (Aqua color™)



50

In-line Placement of the PMV® 2001 (Purple color™)



51

Apply Pilot Balloon Warning Label



52

Step 5: Place the Valve In-line and Assess the Patient

53


Factors Affecting Airway Patency

- Tracheostomy tube
- Cuff issues
- Airway obstruction



54


Importance of Tube Position

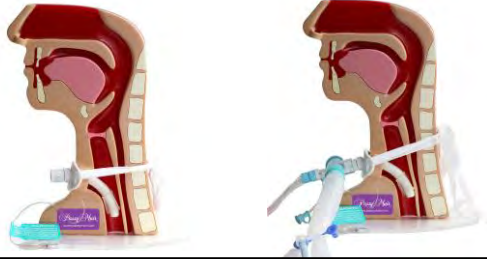

55

Troubleshooting

- PIP increases with each breath
 - Increased WOB is observed
 - High pressure limit alarms
- Valve is removed
 - A loud whooshing sound heard from the tracheostomy tube
- What could cause this?
- Recommendations?

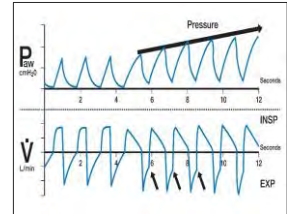

56

Listen for Back Pressure




57

Step 6: Adjust the Vent as Necessary

58

Ventilator Assessment and Adjustments

- Adjust PEEP
- Evaluate sensitivity
 - Pressure vs. Flow Trigger

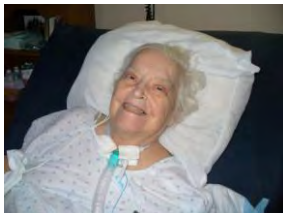

59

Ventilator Assessment and Adjustments

- Volume compensation
 - Increase V_T in small increments to achieve pre-cuff deflation PIP



60

Ventilator Assessment and Adjustments


- Pressure Ventilation
 - May adjust to achieve audible voice and adequate ventilation



61

Ventilator Assessment and Adjustments

- Pressure Support
 - Exp % sensitivity
 - Inspiratory cycle off
 - Set I-time
- Pressure Control
 - Set I-time

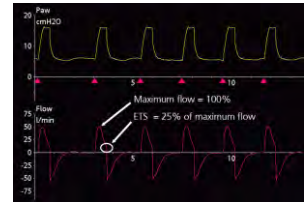

62

Considerations with NIV

- Airway patency assessment should not be done in NIV
- Uses the same settings or as close as possible
- Maybe necessary to do a trial prior to cuff deflation and Valve placement



63

Alarm Settings – Safe Practice

Low exhaled V_t and V_e alarms

Low pressure alarm

- Set 5 to 10 cmH_2O below PIP

High pressure alarm

- Set 10 cmH_2O above PIP

High respiratory rate

- 10 to 15 above baseline



64

Humidification

- Heat/Moisture Exchanger (HME) is ineffective
- Use with Heated Systems
- Remove PMV for medicated treatments



65

Ventilator Settings and Alarm Management



66

Case Study



67

Gil

- Ventilator settings:
 - A/C RR 8
 - V_T 700 cc
 - PIP 35 cmH_2O
 - PEEP 5 cmH_2O
 - F_{O_2} .28
- Tracheostomy
 - 1 month
 - Size 8 Shiley XLT



68

Cuff Deflation Assessment

- Adjust PEEP
- Slow cuff deflation
- Ventilator:
 - Exhaled V_T 300 cc
 - PIP 12 cmH_2O
- Patient:
 - Weak cough
 - Voicing
- Should the Valve be placed in-line?



69

Vent Changes Increase Success


- Valve is placed in-line
- Assessment reveals:
 - Whispers only
 - Poor chest expansion
 - Increased RR
- What ventilator change could be made?



70

Vent Changes Increase Success

- Vent change:
 - Increase V_T to meet but not exceed pre-cuff deflation PIP

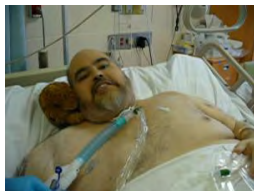

71

Gil



72

Breakout Sessions: Ventilator Application and Mock Assessments



73

VITO Demonstration



74

Hands-On: Parts and Pieces



75

Mock Assessments



76

Mock Assessment: Stacie




77




What would you do next?


78

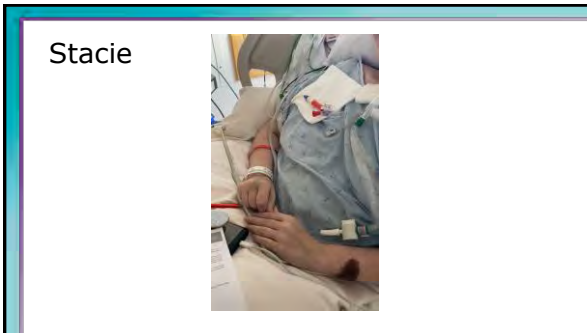


79

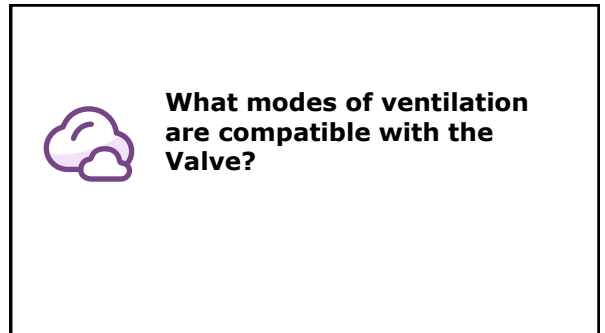


80

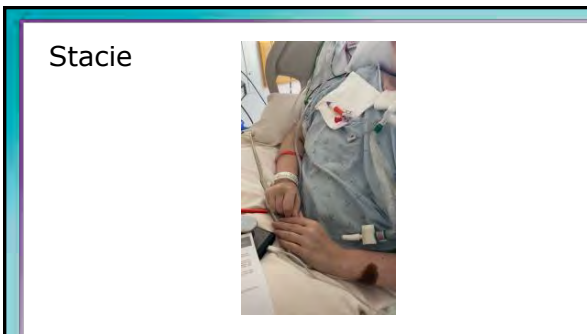


81

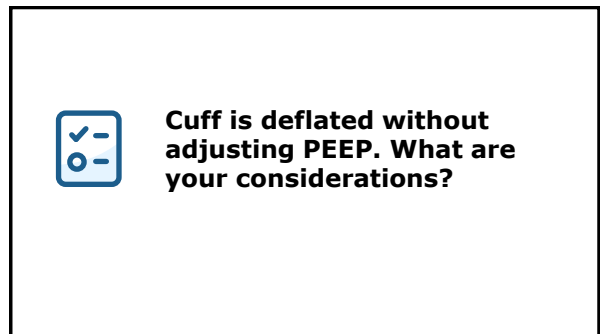

82

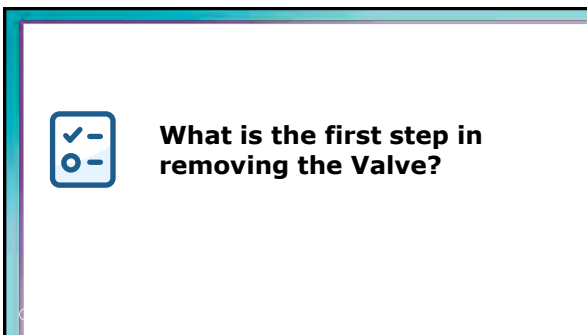


83

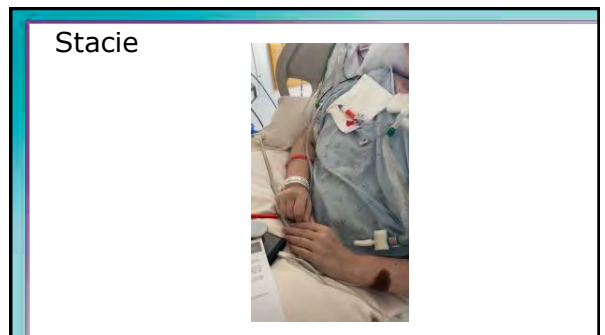

84

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Mock Assessment: 2



85



What issues did you see?

86



87



What would have been the correct order in removing the Valve?

88

Trachlore, Barriers, and More

- Panel Discussion – starting point:
 - You have to wait until a patient is weaned from the ventilator.
 - Our patients are too sick to use a Valve.
 - You need a fenestrated tracheostomy tube.
 - We have to keep the cuff inflated due to aspiration.
 - My patient cannot tolerate cuff deflation trials, so they are not ready for a Valve.
 - My patient speaks with a leak, so a Valve is not needed.
- What have you heard?

89



Audience Q&A

90

Receiving CEUs for this Course

- You will have 5 days from the time this course ends to complete the evaluation, which is required to receive credit
- Go to: <https://ep.passy-muir.com>
- Login or create an account
- Click on the purple box
 - Upper righthand corner
 - Labeled "Enter Meeting Code Here"
- The meeting code is:

91